
**ULINE** S-20903, S-20904  
S-20905  
**ACE™ ELASTIC BANDAGE**

1-800-295-5510  
uline.com


## INSTRUCTIONS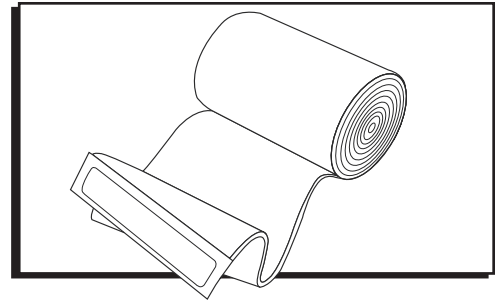


**IMPORTANT!** Before using, you must re-roll the bandage to ensure the fastener is in the proper place for wrapping.

### PREPARATION


1. Disconnect the hook closure from bandage. (See Figure 1)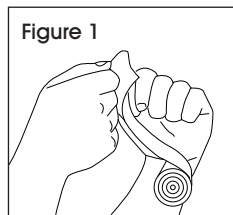


2. Hold bandage so ACE™ label is facing away from you. (See Figure 2)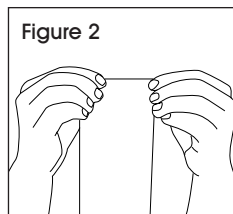

3. Begin re-rolling bandage so the ACE™ label is on the inside. (See Figure 3)

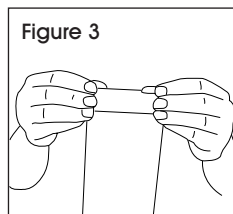
4. With bandage re-rolled, the fastener end will be on the correct side to fasten the bandage after wrapping. (See Figure 4)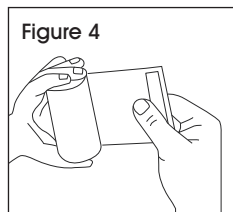


### FOOT/ANKLE


1. Start wrapping at ball of foot, continuing under foot from inside to outside. (See Figure 5)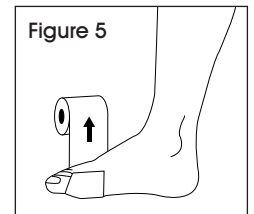


2. Make two or three turns around foot, moving towards ankle with each turn. Overlap one-half of the previous layer. Begin a figure-eight turn bringing bandage up outside of foot, over instep and inside around ankle. (See Figure 6)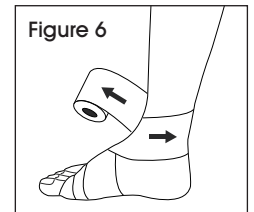

3. Continue down inside of foot around heel, back up over instep, down under foot, and back up completely around ankle. (See Figure 7)

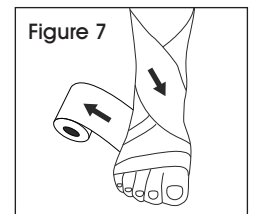
4. Repeat figure-eight two or three times, overlapping one-half of the previous layer and rising above the ankle. (See Figure 8)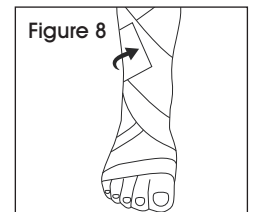

## CARE

1. Rinse the bandage in lukewarm soapy water by hand. Please note, the antimicrobial treatment lasts for at least 10 washings. To prolong the life of the bandage, do not scrub the bandage.
2. Dry the bandage by laying it out flat on a clean surface. After the bandage has dried, roll the bandage as you would after using it. Be sure to smooth out all wrinkles. This is essential for re-bandaging. No ironing is required.

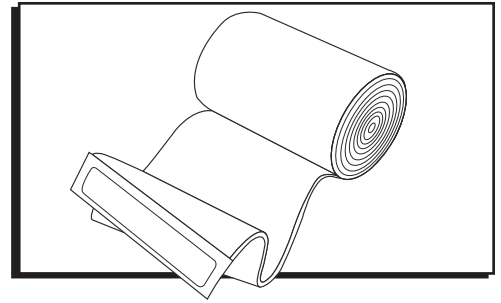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