3M™ Health Care Particulate Respirator and Surgical Mask 1870

User Instructions

IMPORTANT: Keep these User Instructions for reference

W WARNING

This respirator helps protect against certain particulate contaminants but does not eliminate exposure to or the risk of contracting any disease or infection. **Misuse may result in sickness or death.** For proper use, see supervisor or User Instructions or call 3M Health Care Helpline at 1-800-228-3957. In Canada, call 3M Helpline at 1-800-563-2921.

IMPORTANT: Before use, respirator must read and understand these User Instructions. Keep these User Instructions for reference.

- Place respirator on your face so that the foam rests on your nose and the bottom panel is open under chin. Pull the top strap over your head and position it high on the back of the head (Fig. 3a). Then pull the bottom strap over your head and position it around the neck and below the ears (Fig. 3b). Adjust for a comfortable fit by pulling top panel toward the bridge of the nose and bottom panel under chin.
- Place your fingertips from both hands at the top of the metal nosepiece. Using two hands, mold the nose area to the shape of your nose by pushing inward while moving your fingertips down both sides of the nosepiece (Fig. 4).
- W Pinching the nosepiece using one hand may result in improper fit and less effective respirator performance. (Use two hands).
- Perform a User Seal Check (Fig. 5). To check the respirator-to-face seal, cover the middle panel with one or both hands. Inhale and exhale sharply. Be careful not to disturb the position of the respirator. If air leaks around the nose, re-adjust the nosepiece as described in Step 4. If air leaks around respirator edges, adjust panels and position of straps and make certain respirator edges fit snugly against the face. **If you CANNOT achieve a proper seal, DO NOT enter the contaminated area. See your supervisor.**

Removal Instructions:


- Without touching the respirator facepiece, slowly lift the bottom strap from around your neck up and over your head (Fig. 1).
- Then lift off the top strap (Fig. 2).
- Store or discard according to your facility's infection control policy (Fig. 3). Dispose of used product in accordance with applicable regulations.

Storage Conditions and Shelf Life:

Before use, store respirators in the original packaging, away from contaminated areas, dust, sunlight, extreme temperatures, excessive moisture and damaging chemicals. When stored in original packaging between temperatures from -4°F (-20°C) to +86°F (+30°C) and not exceeding 80% RH, the product may be used until the date specified on packaging located next to the "Use by Date" symbol.

- = Use by Date
- = Manufacturer
- = Date of Manufacture
- = Manufacturer's Lot Number relevant to the device bearing the symbol

NIOSH Approved N95 Respirator:
At least 95% filtration efficiency against solid and liquid aerosols that do not contain oil.

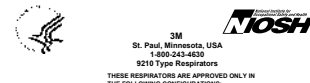TC	PROTECTION	RESPIRATOR COMPONENTS					CAUTIONS AND LIMITATIONS*
		0210	0211	1870	8870F	8870F	
84A-2980	N95	X					ABC, MPOC†
84A-2980	N95		X				ABC, MPOC†
84A-3844	N95			X			ABC, MPOC†
84A-4516	N95				X		ABC, MPOC†
84A-4517	N95					X	ABC, MPOC†

- PROTECTION**
N95: Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil. Use restrictions may apply.
- CAUTIONS AND LIMITATIONS**
A - Not for use in atmospheres containing less than 19.5% oxygen.
B - Not for use in atmospheres immediately dangerous to life or health.
C - Do not exceed maximum use concentrations established by regulatory standards.
J - Failure to properly use and maintain this product could result in injury or death.
M - All approved respirators shall be selected, fitted, used and maintained in accordance with NIOSH, OSHA and other applicable regulations.
N - Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
O - Refer to User's Instructions, and/or maintenance manuals for information on use and maintenance of these respirators.
P - NIOSH does not evaluate respirators for use as surgical masks. Rev. F: 10-18-07

FOR MORE INFORMATION

and assistance on 3M Health Care products, contact your local 3M representative or call 3M Health Care Helpline toll free in USA 1-800-228-3957. In Canada, call 3M Helpline at 1-800-563-2921.

Made in UK by:

3M Health Care
2510 Conway Ave.
St. Paul, MN 55144
1-800-228-3957
www.3m.com/healthcare
© 3M 2013. All rights reserved.
3M and the respirator shape are trademarks of 3M Company, used under license in Canada.

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Fitting Instructions: Must be followed each time respirator is worn.

- Remove respirator from packaging and hold with straps facing upward (Fig. 1a). Place the bottom strap under the center flaps next to the "ATTENTION" statement (Fig. 1b).
- Fully open top and bottom panels, bending nosepiece around thumb at center of foam. Straps should separate when panels are opened. Make certain bottom panel is unfolded and completely opened (Fig. 2).

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