
ULINE S-22070

3M™ COBAN™ 3" WRAP

1-800-295-5510
uline.com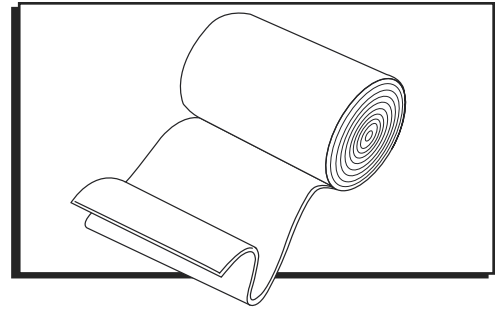

INSTRUCTIONS

USE

Coban™ self-adherent wrap is intended for use as an elastic wrap to provide compression or support, or to secure dressings or devices. This product is not designed, sold or intended for use except as indicated.

STOP **WARNING!** Wrap will not slip or loosen after application. To avoid impairing circulation and causing serious injury, do not wrap too tightly. The risk of impaired circulation is higher for children.

STOP **WARNING!** Monitor area frequently (no less than every 24 hours) for signs of swelling, discoloration, pain, numbness, tingling or other changes in sensation. If these symptoms occur, remove wrap immediately and contact your healthcare provider.

STOP **WARNING!** Do not place wrap in direct contact with an open wound, as this may lead to infection.

! **CAUTION!** This product contains natural rubber latex that may cause allergic reactions.


! **CAUTION!** When used as a component in a compression wrap system, it is important to ensure adequate arterial blood flow to avoid injury related to decreased circulation. Wrap should be used as a component of a compression wrap system only under the supervision of a wound care specialist.

! **CAUTION!** Do not reuse. Reuse may result in compromising product integrity and/or lead to device failure.


DIRECTIONS FOR USE

1. Unwind approximately 12 inches of wrap from the roll and allow the wrap to relax.


2. Without stretching, apply one full wrap and overlap. Press overlapped area lightly to keep the end in place. (See Figure 1)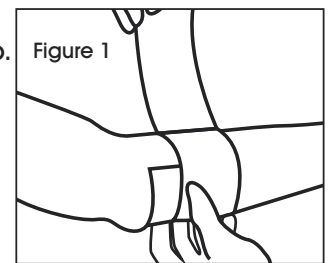

3. Proceed with wrapping, applying tension to the wrap only to obtain desired compression. Always unwind more wrap from the roll before making a wrap so that the product is not applied with too much tension. (See Figure 2)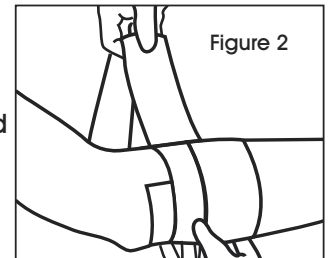

4. When application is complete, cut off excess wrap. Secure the end of the wrap by gently pressing.

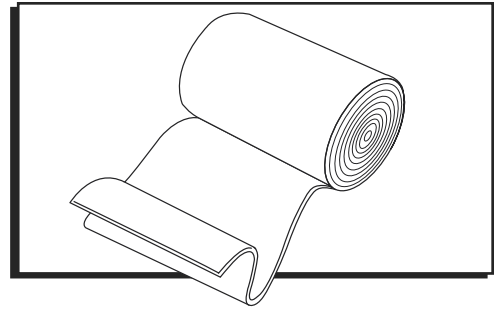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